Introduction

Saudi Food and Drug Authority, represented by National Center for Medical Devices Reporting (NCMDR), would like to warn all consumers and healthcare practitioners that Abbott Diabetes Care company has recalled its Freestyle InsuLinx blood glucose meters because they malfunction at extremely high blood glucose levels.

Affected Products

Freestyle InsuLinx blood glucose meter (Home use)

Manufacturer

Abbott Diabetes Care company

Nature of the problem

SFDA has become aware through **Abbott Diabetes Care** that they are recalling and replacing all **Freestyle InsuLinx** glucose meters because:

- 1- The meter display and store inaccurate low results at rare, extremely high blood glucose levels of 1024 mg/dL (56.88 mmol/l) and above .
- 2- There may be a delay in the diagnosis and treatment of severe hyperglycemia or incorrect treatment may be given.

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SFDA assures that the device mentioned above did not obtain a marketing authorization from SFDA and haven't been cleared by SFDA via Saudi Arabia ports of entry.

Recommendations

Saudi Food and Drug Authority recommends all consumers and healthcare practitioners to adhere to the following :

- Discontinue the use of Freestyle InsuLinX Blood Glucose Meters and use another meter for testing blood glucose.
- Call Abbott Diabetes Care customer services at **800 124 1238 Ext. 425** to receive a replacement meter at no charge if obtained by any other means.
- Inform the Saudi Food and Drug Authority if you have affected product through NCMDR website: <http://ncmdr.sfda.gov.sa>

Saudi Food and Drug Authority encourages all to report any adverse event associated with medical devices through the national center for medical devices reporting

<http://ncmdr.sfda.gov.sa>

For more information please visit the Saudi Food and Drug Authority website :

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<http://www.sfda.gov.sa/En/MedicalEquipments>

Affected products images (Appendix 1)

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